

INSTRUCTIONS

Setting Up Your Safe Place Bed



1

Each **bundle** comes with a durable duffel bag.

(Duffel model may vary)

The **bed ONLY** option comes only with bed and straps.



2

Open bag, remove air pump and set aside for now. Take out bed and place on stationary bed that is a full (74"Wx54"L) or larger. Unfold bed long-ways on stationary bed.

Make sure you can see all of the netting on door closest to you. Inflation point should be on your lower left/right corner of bed depending on bed placement.




3

Use wipes or slightly dampened towel to wipe off bottom outer part of white valve before placing it in clear plastic hole. Place white one way valve into clear inflation point opening.



4

Make sure white valve is fully pushed into clear opening, there should be no gap between top of clear plastic and white valve. Repeat step 3 and 4 on inflation valve's located at base, head and bilateral long sides of bed.

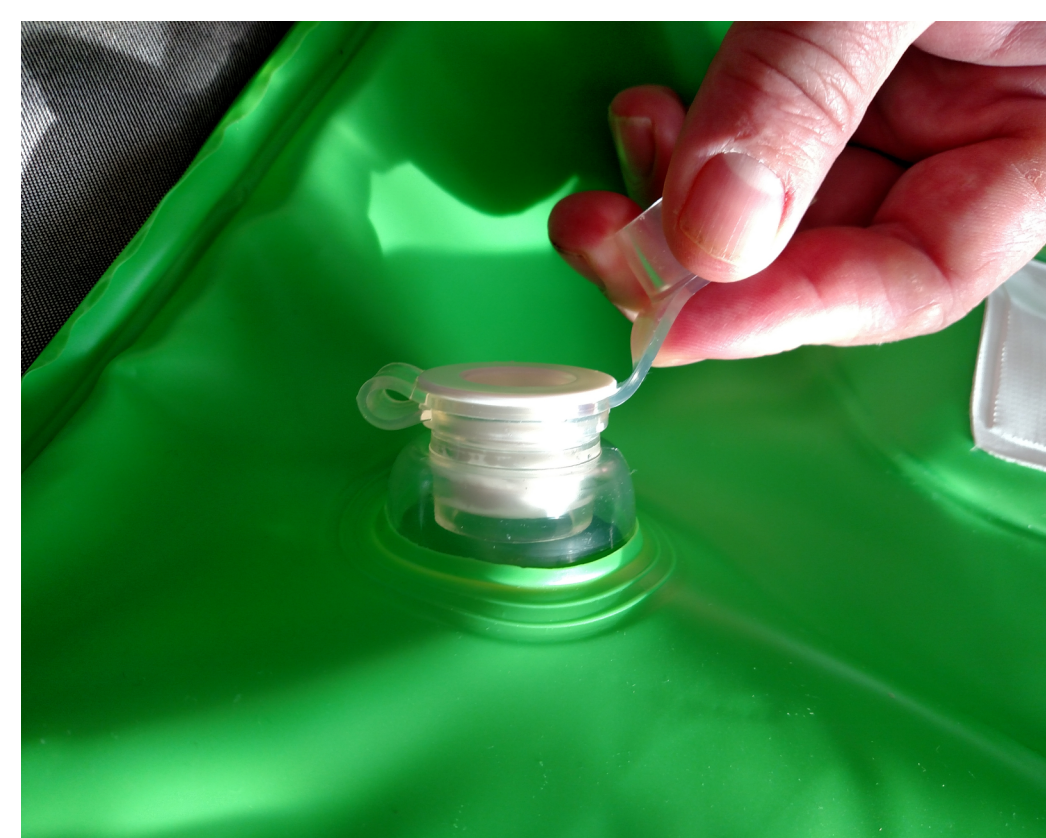

5

Prepare your chosen air pump.



6

Place tip of air tubing into white valve at inflation point, you may need to hold tubing in white valve during inflation, inflate until wrinkles are out of plastic. Inflated chamber should collapse easily when squeezed by your hand. **Be careful not to over inflate bed this will void warranty!** Move on to inflate head or base, once again **do not over inflate**. Tent should be erect at this point.



7

Your bed should look like picture to the right, place inflatable bed in center of stationary bed with equal distance on both sides from edge of bed.



Caption

INSTRUCTIONS

Setting Up Your Safe Place Bed



8

Push clear plugs into white one way valves, then depress valve assembly into bed, repeat on all four valves. The valve will be flush with the sides of the bed.



10

Tighten straps at each buckle till bed is not able to move or tip.

Latch must be connected to top zipper with second zipper under it to be considered locked, please inflate and deflate bed with doors closed and locked.



Reverse directions to take down.

9

Remove straps from duffel bag. Strap Safe Place bed to stationary bed: Connect long strap to strap attached to inflatable bed at the buckle (located directly below netting door on long side), loop strapping under bed frame to opposite side of bed and reconnect to inflatable bed, repeat this step on opposite side of bed. Every time you set up bed please inspect strapping and buckles for damage.

The Safe Place Bed may also be attached to an inflatable air mattress that is a queen size or larger, or you may place a sleeping mat inside. To see what we recommend please visit safeplacebedding.com to view our sleeping mat in the store.



Daily Use

- Before placing child/user in bed nightly do a visual inspection of netting to look for holes starting, if any seen, stop using bed and contact customer service.
- Do a daily manual inspection by squeezing each air chamber of bed (4 chambers total) making sure it is still inflated, if one chamber is deflated take out clear plug and white one way valve from clear plastic opening/inflation point and follow steps 4-8 from instruction manual. Do not use bed for 24 hours to make sure chamber will hold air. If chamber deflates again contact customer service.
- Do a manual inspection of straps by pulling on each strap with hand to make sure straps are tight and bed remains in center of full or larger bed.
- Do a manual inspection of zipper as you close door of bed if you see a problem do not use bed and contact customer service.
- Clean bed with warm water and very mild detergent on cloth if needed. We highly recommend using an incontinence pad to help keep bed clean. You can find these on our store at safeplacebedding.com.

WARNINGS

Stay Safe While Using Your Bed



Risk of suffocation if used before 36 months old or low muscle tone. Please consult your Doctor before using product.



- Always consult your medical Doctor before using this product.

- Keep away from water, not a flotation device.

- If one side of bed deflates stop using bed.

- Firmly attach to stationary bed when in use, not tested independently.

- Sleeping mat is not a tested accessory, speak with your medical doctor and physical therapist before use.

- Safe Place Bed is twin (39" W Xx75" L) in size, must be used on a full size bed (74"Wx54"L) or larger bed.

-

- This product is not a toy. Monitor user closely when he/she has access to outside of bed, risk may include but not limited to strangulation from straps, choking on zipper, and entrapment .



No smoking or heat source around product, flammability has not been tested.

Keep sharp objects away from product, punctures will not be covered in warranty and may cause injury to child.

